



LookinBody Web

Unsaved Results - Temporary Database



Introduction

Potentially mismatched results are flagged as unsaved and automatically sent to a Temporary Database.

Such results require review for corrections before they can be saved. Additionally, these results will not be available on the InBody Mobile App until they have been reviewed and saved through [LookinBody web](#).

Items you will need:




- Access to a computer/tablet with internet connection
- Compatible browser such as: Internet Explorer, Google Chrome, Firefox, or Safari

Estimated Time to Complete: 15 Minutes

When Will a Test be Marked for Review?

After a test is taken, the Temporary Database system compares the member profile used to take the test to the member profile saved in the Member List.

The following inconsistencies flag test results as unsaved:


-  1 year+ age difference
-  1 inch+ height difference
-  Mismatched ID and/or Mobile Number

Review & Edit Unsaved Results



Update the Temporary Database

1. Use any browser to go to the LookinBody Web Portal: <https://usa.lookinbody.com/>
2. Log in to your account.

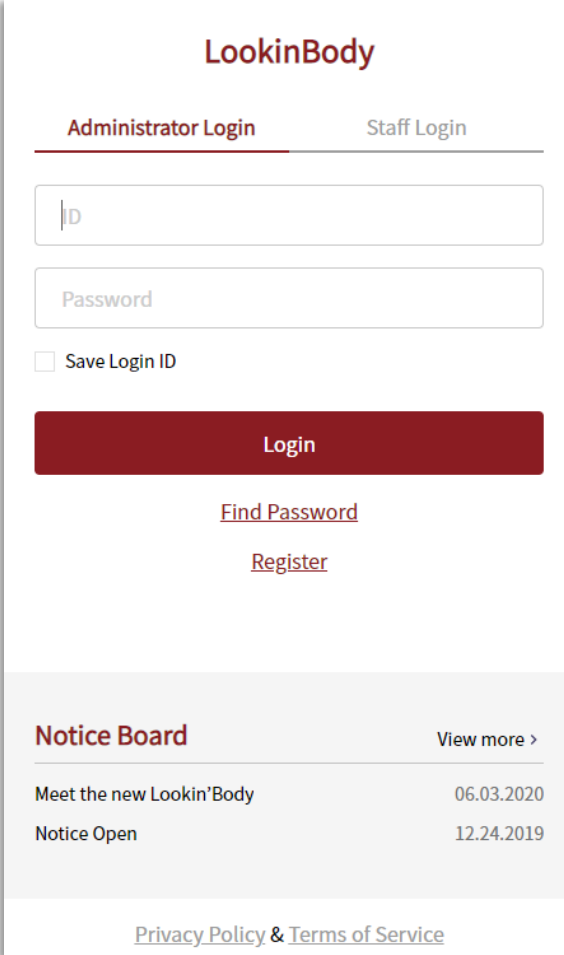

The screenshot shows the LookinBody login interface. At the top, the 'LookinBody' logo is displayed. Below it, there are two tabs: 'Administrator Login' (which is selected) and 'Staff Login'. The login form includes a text input for 'ID', a text input for 'Password', and a checkbox labeled 'Save Login ID'. A prominent red 'Login' button is positioned below the password field. Underneath the button are two links: 'Find Password' and 'Register'. At the bottom of the page, there is a 'Notice Board' section with a 'View more >' link. The notice board contains two entries: 'Meet the new Lookin'Body' dated '06.03.2020' and 'Notice Open' dated '12.24.2019'. At the very bottom, there is a link for 'Privacy Policy & Terms of Service'.

LookinBody

Administrator Login Staff Login

ID

Password

☐ Save Login ID

Login

[Find Password](#)

[Register](#)

Notice Board [View more >](#)

Meet the new Lookin'Body	06.03.2020
Notice Open	12.24.2019

[Privacy Policy & Terms of Service](#)

Review Unsaved Results

Adjust the number of displayed **Unsaved Results** by clicking the drop-down menu

55555555 Search Filter InBody Test: 07.23.2020. - 07.30.2020. Add Filter

2 / 2 people (0 people selected) Show Unsaved Results: Not more than 10 Register New Edit Multiple Set Column Excel Download View 30

	Name	User ID	Mobile No.	Edit	Report	Access to Home-use Product Data	Email	Age	Gender	Height	Registration Date	Last InBody Test Date	Group
!		5555555555	55555555**					45.0	Male	6ft.01.0in.		06.22.2020	
!		5555555555	55555555**					25.0	Male	6ft.01.5in.		06.03.2020	
!		5555555555	55555555**					40.0	Male	6ft.02.0in.		05.13.2020	
!		5555555555	55555555**					41.0	Male	6ft.01.0in.		05.13.2020	
☆	Test Profile	5555555555	55555555**			Request	lbw0eb@inbody.com	40	Male	6ft.01.0in.		08.03.2020	Test Group
☆	Clark Kent	0991	55555555**			Request	lbweb@inbody.com	37	Male	5ft.09.0in.		12.29.2017	March Madness Fitness Challeng

Temporary Data

!

A Test Result Needs Your Attention

- InBody Test Date & Time: 08.04.2020 14:39
- Please review the test by displaying **Unsaved Results** in the **Member list** page

OK

If you are logged into the web portal during a test, a message indicating the need for test review will be displayed.

Review Unsavd Results

The Member List page displays **Unsaved Results** and will be highlighted in red

Click on any highlighted result to review and save.

Search

☐ Filter

InBody Test:

07.23.2020. - 07.30.2020.

Add Filter

2 / 2 people (0 people selected)

Show Unsavd Results:

Not more than 10

Register New

Edit Multiple

Set Column

Excel Download

View 30

<input type="checkbox"/>	Name	User ID	Mobile No.	Edit	Report	Access to Home-use Product Data	Email	Age	Gender	Height	Registration Date	Last InBody Test Date	Group
		5555555555	55555555**					45.0	Male	6ft.01.0in.		06.22.2020	
		5555555555	55555555**					25.0	Male	6ft.01.5in.		06.03.2020	
		5555555555	55555555**					40.0	Male	6ft.02.0in.		05.13.2020	
		5555555555	55555555**					41.0	Male	6ft.01.0in.		05.13.2020	
<input type="checkbox"/>	Test Profile	5555555555	55555555**			Request	lbw0eb@inbody.com	40	Male	6ft.01.0in.		08.03.2020	Test Group
<input type="checkbox"/>	Clark Kent	0991	55555555**			Request	lbweb@inbody.com	37	Male	5ft.09.0in.		12.29.2017	March Madness Fitness Challenge

Update the Temporary Database

After selecting a highlighted test result, a comparison sheet is displayed

- Unsaved Result (shown left)
- The saved web portal profile (shown right).

Review the data in the highlighted fields.

Unsaved Results

When the testing profile does not match the member info, the result is tagged for review. Please review the date to make any corrections or it can also be deleted.

Unsaved Results

InBody Test Date and Time
06.03.2020. 15:20

User ID
5555555555

Mobile No.
5555555555

Height
6 ft. 1.5 in.

Date of Birth
▼ Month Day Day 1995 Year 25

Gender
☐ Female
☒ Male

Existing Member Information

Name
Test Profile

User ID
5555555555

Mobile No.
5555555555

Height
6 ft. 1 in.

Date of Birth
Jan ▼ Month 01 Day 1980 Year

Gender
☐ Female
☒ Male

Delete

Cancel

Next

Update the Temporary Database

Click “Next” to edit profile information.

You can also click on “Delete” to permanently erase unwanted results.

Unsaved Results

When the testing profile does not match the member info, the result is tagged for review. Please review the date to make any corrections or it can also be deleted.

Unsaved Results

InBody Test Date and Time

06.03.2020. 15:20

User ID

5555555555

Mobile No.

5555555555

Height

6

ft.

1.5

in.

Date of Birth

▼

Month

Day

Day

1995

Year

25

Gender

Female

Male

Existing Member Information

Name

Test Profile

User ID

5555555555

Mobile No.

5555555555

Height

6

ft.

1

in.

Date of Birth

Jan

▼Month

01

Day

1980

Year

Gender

Female

Male

Delete

Cancel

Next

© 2020 InBody Confidential and Proprietary

InBody

Updating the Temporary Database

Select which profile information you would like to keep

Selecting the **Existing Member** information will automatically update the test result.

Click **OK** to confirm your selection.

The screenshot shows a software interface with a modal dialog titled "Unsaved Results". The dialog contains the following text and options:

- Header: Unsaved Results
- Message: "When the testing profile does not match the member info, the result is tagged for review. Please review the date to make any corrections or it can also be deleted."
- Section: "Unsaved Results" (highlighted in red)
 - ☒ Unsaved Results
 - Height: 6ft. 01.5in.
 - Date of Birth: (25.0)
- Section: "Existing Member Information" (highlighted in red)
 - ☐ Existing Member Information
 - Height: 6ft. 01.0in.
 - Date of Birth: 01.01.1980 (40)
- Buttons: "Previous" (disabled), "OK" (active), "Cancel", "Next" (disabled)

The background interface shows a form with fields for "InBody Test Date and Time" (06.03.2020. 15:20), "User ID" (5555555555), "Mobile No." (5555555555), "Height" (6 ft. 1.5), "Date of Birth" (Month dropdown, Year dropdown), and "Gender" (Female, Male radio buttons).

Additional Notes



Additional Notes

Unsaved Results and/or test results sent to the Temporary Database are only accessible through web portal, and they are unavailable through the InBody App.

To reduce the possibility of a test being flagged for review:

- 👤 Verify that the unit is connected and logged in before testing.
- 👤 Set up a member's profile with the correct Date of Birth.
 - The member's age will automatically update every year.
- 👤 Encourage the member to review their profile information before testing.
- 👤 Encourage members to notify staff members of profile information updates.

For any issues or questions, please feel free to contact the support team for assistance.

LookinBody Web
Support Team



LBWeb@InBody.com



323.932.6503 extension 2



www.InBody.com

